

CORONAVIRUS



MedicalSystem

Coronavirus Disease 2019 Antibody (IgG/IgM) Combined Test Kit



RAPID

Results within 5-10 minutes

DIVERSE

Serum, Plasma, Whole blood

EASY

Only 4 steps taken

RELIABLE


IgG/IgM tested at one time

Specification

Product name	Coronavirus Disease 2019 Antibody (IgM/IgG) Combined Test Kit
Test principle	Lateral flow immunoassay
Sample	Whole blood, Serum, Plasma
Time to report	5-10minutes
Operation temperature	Room temperature
Pack	25 tests
Material provided	25 individual sealed pouches, 25disposable droppers

MedicalSystem's IgM/IgG antibody combined rapid test for COVID-19 is used to qualitatively detect IgG and IgM antibodies of the novel coronavirus (SARS-CoV-2) in human serum, plasma or whole blood in vitro. These biomarkers indicate that a person's immune system has responded to infection with the COVID-19 virus.

This innovative rapid test kit is developed with lateral flow immunoassays technology in high sensitivities and specificities. It is quite easy to use and ideal for screening check in epidemics control of COVID-19 virus.

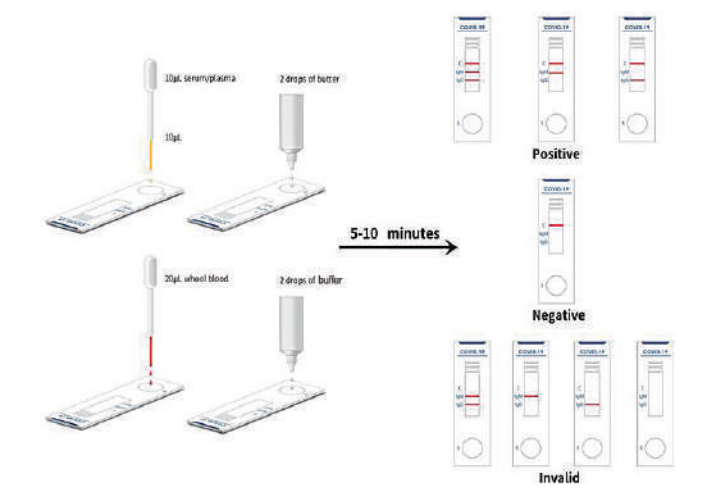

1. Collect blood sample
2. Add 10 µL serum/plasma or 20 µL whole blood to sample well
3. Place 2 drops of buffer in sample well
4. Read results in 5-10 minutes

MedicalSystem Biotechnology Co., Ltd.

ADD: No.299, Qiming South Road, Yinzhou, Ningbo 315104, P.R.China

Tel: +86-574-55337009 Fax: +86-574-88231889 E-mail: sales@nbmedicalsistem.com